VIZMAXX® Autofocus sunglasses

Made in China / Fabricado en China / Fabriqué en Chine
Hergestellt in China / Made in Cina
Fabricado na China / Vervaardigd in China

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• ENGLISH •

WARNINGS

IMPORTANT: Vizmaxx® Autofocus glasses are not intended to replace corrective glasses prescribed by your ophthalmologist, nor should they replace regular eye examinations. Have your eyes tested regularly by an eye care professional to ensure that you maintain good vision. Eye examinations can detect health problems that have no signs or symptoms. Have your eyes checked regularly. Never disregard or delay getting professional eye health advice because of the purchase, receipt or use of Vizmaxx® Autofocus. Do not look directly at the sun. Looking at or near the sun will cause immediate and irreversible damage to your eyes. DO NOT use solvents or products containing alcohol to clean Vizmaxx® Autofocus. As with any multifocal lens, the eye will need a period of time to adapt and achieve optimal vision.

 Not suitable for driving

The set includes:

1 pair of Vizmaxx® Autofocus glasses
1 microfibre pouch
1 instruction manual

USE

Vizmaxx® Autofocus glasses are suitable for helping you improve your near and middle-distance vision. The lenses of the Vizmaxx® Autofocus glasses are multifocal and provide correction ranging from 0.5 to 3 dioptres. Multifocal lenses mean that you don't need to change your glasses constantly. Simply adjust the point of vision depending on your correction needs. For optimal vision, simply look through the lens at the point you see most clearly.

CLEANING

To avoid scratching the lenses, we recommend that you first wash the glasses with tap water. Then clean your Vizmaxx® Autofocus glasses with a microfibre cloth and a special lens cleaner. The microfibre pouch that came with your Vizmaxx® Autofocus glasses can also be used as a cloth to clean the lenses.

MAINTENANCE

We recommend that you put on and remove your glasses gently and carefully, using both hands, to avoid bending the frames. We also advise against placing your glasses on your head, as this could distort the frames. Intense heat can also distort the frames, so we recommend that you do not leave your glasses on the dashboard or car seat. To avoid scratching the lenses, always place your glasses on a flat surface with the lenses facing up to ensure that the lenses do not come into contact with any surface.

Quality Warranty:

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country. This warranty does not cover damages resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

Australia & New Zealand only

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries.

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