
# Ear Wizard®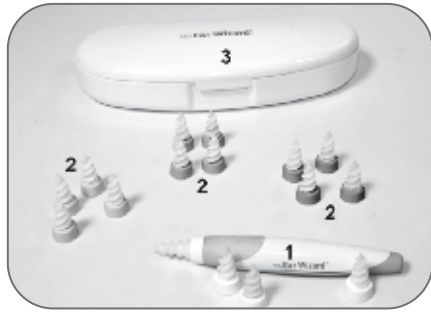

## English

Thank you for purchasing Ear Wizard®, the ear cleaner with silicone tips that enables you to clean your ears comfortably and safely. To ensure you use the product correctly, read all these instructions before using it for the first time.

### Product Description

- 1.Handle
- 2.Tips
- 3.Case

### WARNINGS

Always read the instructions before using Ear Wizard®. Consult your paediatrician before using Ear Wizard® on children under 12 years of age. This product should not be used on anyone wearing ear tubes. Stop using the product if you feel pain or discomfort, or notice pressure, blood or a loss of hearing in the ears.

### WARNING

Ear Wizard® should not be used in the following situations without medical supervision:

- Anyone suffering from Ménière's disease or who experiences any form of dizziness, vertigo, hearing loss or other ear problems.
- Anyone with an ear infection.
- Anyone wearing ear tubes.
- Anyone who has had mastoid surgery.
- Anyone who has or has had a perforated eardrum.

### WARNING

Before they are reused, the tips (2) should be cleaned and disinfected with soap and water after each insertion in the ear canal. We recommend using a new tip (2) to clean each ear canal. To clean the handle (1), wash it with soap and hot water, and then dry it before storing it.

### WARNING

DO NOT PROBE, PUSH OR FORCE EAR WIZARD® INTO YOUR EAR. THIS PRODUCT IS NOT INDICATED FOR DEEP INSERTION IN THE EAR CANAL. EAR WAX BUILDS UP NEAR THE ENTRY TO THE EAR CANAL. IMPACTED EAR WAX SHOULD BE REMOVED BY A DOCTOR. ALWAYS TURN EAR WIZARD® IN THE DIRECTION INDICATED BY THE ARROW ON THE HANDLE (1). OTHERWISE, SERIOUS INJURY MAY OCCUR.

### INSTRUCTIONS FOR USE

IMPORTANT: Before using Ear Wizard®, notice the arrow on the EAR WIZARD® handle (1). This arrow indicates the direction the handle (1) should always be turned. It should not be turned in any other direction. Using the handle (1) incorrectly may cause serious injury.

- 1.Insert a tip (2) into the handle (1) and turn as shown in Figure D to attach.

Note: Each tip (2) has a coloured base in a total of four different colours. This helps identify them so they can be used by as many as four different people.

### CLEANING:

- 2.Carefully and gently insert the Ear Wizard® tip (2) into your right ear. Do not push the tip too far into your ear.
- 3.Turn Ear Wizard® in the direction indicated by the arrow on the handle (see Figure B). DO NOT TURN THE DEVICE IN THE OTHER DIRECTION.
- 4.To avoid the risk of injury, do not move while using the device.

### CHANGING TIPS

- 5.Change the tip (2) and use a new one before changing from one ear canal to the other. To remove the tip (2), hold the handle (1) with one hand and turn the tip (2) as shown in Figure E. Remove the tip and put on a clean tip (2) by following instruction 1 above.

### MAINTENANCE

- . Place the tips (2) in soapy hot water until they are disinfected.

### The set includes:

- 1 Ear Wizard® handle
- 16 soft tips
- Travel case

### Quality Warranty:

This product is covered by a warranty against manufacturing defects subject to the time periods stipulated by the legislation in force in each country. This warranty does not cover damages resulting from inadequate use, negligent commercial use, abnormal wear and tear, accidents or improper handling.

### Australia & New Zealand only

This product has the benefit of certain Consumer Guarantees. These are prescribed by the Australian Consumer Law & the New Zealand Consumer Guarantees Act 1993 both of which provide protection for consumers. There is no express warranty for this product in Australia or New Zealand. The above paragraph refers to other countries.

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